



HOW TO USE BD™ ESwab™™ and BD™ Universal Viral Transport


1

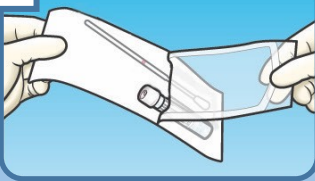

2

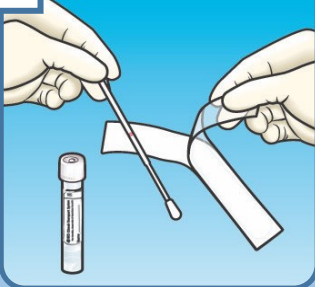

3

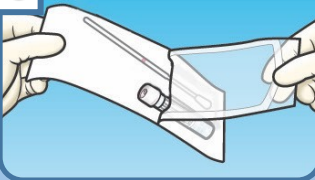


4



5



6

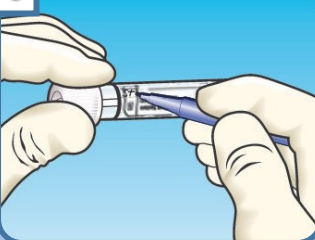

UNC Microbiology Laboratory

101 Manning Drive • Chapel Hill, NC 27514

984.974.1805

Lawson # 054897 and 143675

BD Cat No.: 220246 and 220532

BD ESwab Collection Kit

Blue, Green, or White Cap w/ liquid amies and swab.

For the collection and transport of clinical specimens containing aerobes, anaerobes, and fastidious

bacteria. *NOTE: Swabs are not the preferred type of specimen for culture. Properly collected tissue or aspirates preferred—send in sterile cup.

Anaerobic | Aerobic Wounds, Ear cultures, Eye Cultures, Fungal Cultures, MRSA, VRE, Sinus Cultures, VRE, Yeast Screens, CF Sputum (deep pharyngeal), Rapid Grower Screen, Bordetella pertussis, Bordetella parapertussis



Lawson # 050542

GenProbe No.: 301041

Hologic Aptima Unisex Collection Kit

White cap with FOIL top, white cleaning swab and blue collection swab. *Do not puncture foil top.

For the collection and transport of clinical specimens containing:

Chlamydia trachomatis and Neisseria gonorrhoeae by Nucleic Acid Amplification.

NOT SUITABLE FOR CULTURE!



5



Lawson # 050750

BD Cat No.: 220531

BD Universal Viral Transport

With flexible flex mini-tip flocced swab.

For the collection and transport of clinical specimens containing viruses.


Respiratory Panels [Flu, Covid, RSV, Etc...], Adenovirus PCR (Eye, Peri-Rectal Swab), Ureaplasma PCR, Mycoplasma Pneumoniae PCR




BD Life Sciences

**HOW TO USE
BD™ ESwab™
and
BD™ Universal
Viral Transport**


1

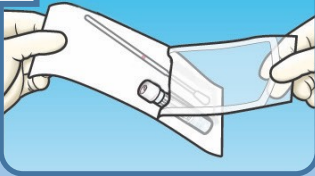

2

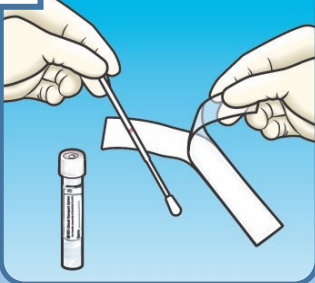

3

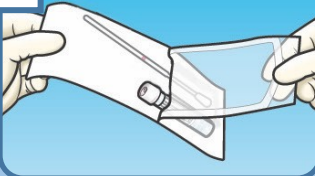


4



5



6

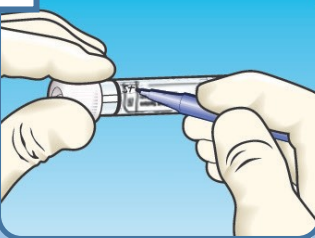

UNC Microbiology Laboratory

101 Manning Drive • Chapel Hill, NC 27514

984.974.1805



Lawson # 050439

Cat No.: 4019

HealthLink Liquid Stuart DoubleSwab

Red Cap two (2) swabs.

For the collection and transport of clinical specimens containing aerobes.

Rapid Strep, Group A Strep Culture

Group B Strep Culture

Vaginitis Screen (BV), Rapid Trichomonas

HSV PCR (Lesion Swabs) - Preferred

VZV PCR (Lesion Swabs) - Preferred

BD Life Sciences

Diagnostic Systems • 7 Loveton Circle • Sparks, MD 21152

Tel: 800.638.8663 • www.bd.com/ds