

Simulation Technology Reference

Link Box
Laerdal



UNC Healthcare
Advanced Life Support Programs
Version 1 Dated 5 June 2013

Link Box

Laerdal



Power supply cable must remain plugged into unit at **ALL** times. This device does not run off of battery power unless utilizing the mobile power cell.

To turn Link Box **ON**, press and hold power button. Device powers off with SimPad or by unplugging power supply

Dual green indicator lights will be present until linked with manikin and SimPad. At that time a solid blue wireless indicator light will appear. Link Box should now ready for use.