

McLendon Laboratories Internal Reference Test Processing Requirements

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB8185	ABN HB CONFIRM	ML SPECIAL HEMATOLOGY	LAVENDER	Whole blood, refrigerate		WBR
LAB43	ACETAMINOPHEN LEVEL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB8005	ACTIVATED PROTEIN C RESISTANCE	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB429	ACYLCARNITINES, PLASMA, QUANTITATIVE	ML SPECIAL CHEMISTRY	GREEN TOP - SODIUM HEPARIN	Centrifuge, aliquot, freeze		CAF
LAB1029	APT	ML SPEC HEMATOLOGY				
LAB5317	ADAMTS13	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB5806	ADENOVIRUS PCR	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, Stool only must be in STAR Buffer		PRTM
LAB5654	ADENOVIRUS PCR BLOOD	ML MOLECULAR MICROBIOLOGY	PEARL TOP - or 2 KEDTA	Centrifuge, aliquot, Refrigerate		AR
LAB559	AFP TUMOR MARKER	ML CHEMISTRY	RED TOP -PLAIN or SST	Centrifuge, aliquot, freeze		AF
LAB45	ALBUMIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB689	ALBUMIN / CREATININE URINE RATIO	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		PR
	ALBUMIN / CREATININE URINE RATIO [1230101599]	REX	24 hour container	refrigerated		PR
	ALBUMIN 24HR URINE [1230100326]	REX	STERILE CONTAINER	refrigerated (Note total volume)		AR
	ALBUMIN RANDOM URINE [1230100327]	REX	STERILE CONTAINER	refrigerated		PR
	ALBUMIN RANDOM URINE [1230100327]	REX	STERILE CONTAINER	refrigerated		PR
LAB177	ALBUMIN, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB410	ALBUMIN, URINE, 24 HOUR	ML CHEMISTRY	24 HR URINE CONTAINER	Centrifuge, aliquot, refrigerate, record time and volume		CAR
LAB5351	ALCOHOL, ISOPROPANOL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB5133	ALCOHOL, METHYL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB5132	ALCOHOL PANEL * Includes: Ethanol-GC, methyl alcohol, isopropanol, ethylene glycol	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB112	ALKALINE PHOSPHATASE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB1424	ALLERGEN AVOCADO IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1507	ALLERGEN CHAETOMIUM GLOBOSUM IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB606	ALLERGEN CLADOSPORIUM	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1657	ALLERGEN LAMB'S QUARTER IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1648	ALLERGEN RED TOP IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB634	ALMONDS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB810	ALPHA-1-ANTITRYPSIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB5316	ALPHA-2 ANTIPLASMIN ACTIVITY	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB19872	ALPHA-1-ANTITRYPSIN, DNA ANALYSIS	ML DNA	YELLOW (ACD) or EDTA	Shipped cold, not frozen. Do not centrifuge. Can be stored refrigerated up to 72 hours prior to testing.		
LAB132	ALT	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB6033	ALTERNARIA ALTERNATA, IGE	ML IMMUNOLOGY	SST-GOLD	Separate, aliquot and freeze		AF
LAB145	AMIKACIN LEVEL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB5124	AMIKACIN LEVEL, PEAK	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB5125	AMIKACIN LEVEL, TROUGH	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB47	AMMONIA	ML CHEMISTRY	GREEN TOP - SODIUM HEPARIN	Plasma must be separated from cells within 15 minutes of collection. Centrifuge, aliquot, freeze		AF
LAB357	AMPHETAMINE, URINE, QUALITATIVE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB48	AMYLASE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB3558	AMYLASE URINE, RANDOM	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB178	AMYLASE, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB360	AMYLASE, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate, record volume and time.		CAR
LAB147	ANA	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB648	ANTI-DNA ANTIBODY, DOUBLE-STRANDED	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	1 mL blood	AF
LAB513	ANTIMITOCHONDRIAL ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	1 mL blood	AF
LAB458	ANTI-NEUTROPHILIC CYTOPLASMIC ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	1 mL blood	AF
LAB5322	ANTIPHOSPHOLIPID PANEL	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB311	ANTITHROMBIN III	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB325	APTT	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB5070	ARTHROPOD IDENTIFICATION	ML MICROBIOLOGY	STERILE CONTAINER	Room Temperature		RT
LAB5066A	ASPERGILLUS AG, BAL	ML IMMUNOLOGY	STERILE CONTAINER	freeze		PF
LAB598	ASPERGILLUS FUMIGATUS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5066A	ASPERGILLUS GALACTOMANNAN AG,BAL	ML IMMUNOLOGY	STERILE CONTAINER	freeze		PF
LAB1311	ASPERGILLUS GALACTOMANNAN ANTIGEN, SERUM	ML IMMUNOLOGY	SST-GOLD	Centrifuge,Freeze,-Cannot be shared		PF
LAB1501	ASPERGILLUS NIGER IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB131	AST	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB6032A	B57 BY FLOW	ML FLOW CYTOMETRY	YELLOW (ACD)	Whole blood, Room Temp, 72 hours from collection		WBRT
	BACTERIAL VAGINOSIS [1234000040]	REX	STERILE CONTAINER	refrigerated		PR
LAB626	BAHIA GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1628	BAKER'S YEAST IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB364	BARBITURATE, URINE, QUALITATIVE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB15	BASIC METABOLIC PANEL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB5656	BCR/ABL1 P190 ASSAY	ML DNA	PURPLE (EDTA) x2 or YELLOW (ACD)	unspun		PR M,S IF EDTA
LAB5392	BCR/ABL1 P210 ASSAY	ML DNA	PURPLE (EDTA) x2 or YELLOW (ACD)	unspun,		PR M,S IF EDTA

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB12562	BCR-ABL1 MUTATION ANALYSIS(TYROSINE KINASE RESISTANCE)	ML DNA	PURPLE (EDTA) x2 or YELLOW (ACD)	unspun,		PR M,S IF EDTA
LAB56501	BCR-ABL1 P190 AND P210 ASSAY	ML DNA	PURPLE (EDTA) x2 or YELLOW (ACD)	unspun,		PR M,S IF EDTA
LAB1633	BEECH (AMERICAN) TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB618	BEEF IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB367	BENZODIAZEPINE, URINE, QUALITATIVE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB368	BENZODIAZEPINE, URINE, QUANTITATIVE	ML SPECIAL CHEMISTRY	STERILE CONTAINER	Aliquot, refrigerate		AR
LAB646	BERMUDA GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB49	BETA 2 MICROGLOBULIN, SERUM	ML SPECIAL CHEMISTRY	SST-GOLD	Centrifuge, aliquot, Refrigerate		APR
LAB5138	BETA 2 TRANSFERRIN, FLUID	ML SPECIAL CHEMISTRY	STERILE CONTAINER	Refrigerate, note specimen source		PR
LAB1179	BETA-2 GLYCOPROTEIN ANTIBODIES	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB5140	BILIRUBIN SUBFRACTIONATION	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB182	BILIRUBIN, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB52	BILIRUBIN, DIRECT	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB5141	BILIRUBIN, DIRECT, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB50	BILIRUBIN, TOTAL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB601	BIRCH (COMMON SILER) TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1374	BK VIRAL LOAD (QUANTITATIVE)	ML DNA	PEARL or EDTA X 2	Centrifuge, aliquot, freeze		CAF
LAB1375	BK VIRUS, DNA, URINE, QUANTITATIVE	ML DNA	STERILE CONTAINER	Aliquot, freeze		AF
LAB923	BORDETELLA PERTUSSIS / PARAPERTUSSIS PCR	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, NP swab		R
LAB1634	BOX ELDER TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1707	BRAZIL NUT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB107501	BRCA1 AND BRCA2 SEQUENCING	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB10748	BRCA1 KNOWN FAMILIAL MUTATION TESTING	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB10747	BRCA1/2 ASHKENAZI JEWISH MUTATION PANEL	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB6903	BRCA1/2 LARGE DUPLICATION AND DELETION TESTING	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB10749	BRCA2 KNOWN FAMILIAL MUTATION TESTING	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB140	BUN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB8320	C. DIFFICILE PCR	ML MICROBIOLOGY	STERILE CONTAINER	Refrigerated. Must be received within 24 hours		PR
LAB152	C3 COMPLEMENT	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB151	C4 COMPLEMENT	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB155	CA 125	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		AF
LAB8018	CACAO (CHOCOLATE) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB706	CAFFEINE LEVEL	ML SPECIAL CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB53	CALCIUM	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB5146	CALCIUM TIMED, URINE	ML CHEMISTRY	STERILE CONTAINER	Unspun, Aliquot and refrigerate, note volume and time		USAR
LAB5145	CALCIUM, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB371	CALCIUM, URINE, RANDOM	ML CHEMISTRY	STERILE CONTAINER	Unspun, Aliquot and refrigerate		USAR
LAB777	CANCER ANTIGEN 19-9	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB619	CANDIDA ALBICANS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5147	CANNABINOIDS, URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB21	CARBAMAZEPINE LEVEL, TOTAL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB5135	CARDIOLIPIN ANTIBODY, IGM/IGG	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB815	CARNITINE, PLASMA	ML SPECIAL CHEMISTRY	GREEN TOP - SODIUM HEPARIN	Centrifuge, aliquot, freeze		CAR
LAB1615	CARROT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB607	CASHEW NUT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB593	CAT DANDER IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB294	CBC	ML HEMATOLOGY	LAVENDER	Whole blood, Refrigerated: 24 Hours		WBR
LAB293B	CBC W/ AUTO DIFF	ML HEMATOLOGY	LAVENDER	Whole blood, Refrigerated: 24 Hours		WBR
LAB851	CCP IGG (Cyclic Citrullinated Peptide)	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	1 mL blood	AF
LAB6009	CD3 TX WORKUP, FLOW	ML FLOW CYTOMETRY	LAVENDAR	Whole blood, Room Temp, 48 hours		WBRT
LAB5923	CD34 ENUMERATION	ML FLOW CYTOMETRY	LAVENDER	Testing performed only on a scheduled basis, Fresh sample required		WBRT
LAB57	CEA	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		AF
LAB5148	CEA, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB703	CERULOPLASMIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1521	CHEESE, CHEDDAR IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB632	CHICKEN (MEAT) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5157	CHILD ALLERGEN PANEL IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5814	CHLAMYDIA/GONORRHOEAE NAA	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Hologic collection devices		PR
LAB59	CHLORIDE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB183	CHLORIDE, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB5149	CHLORIDE, STOOL	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB374	CHLORIDE, URINE, RANDOM	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB375	CHLORIDE, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate, record volume and time.		CAR
LAB376	CHOLESTEROL, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB60	CHOLESTEROL, TOTAL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB62	CK	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB63	CK TOTAL AND CKMB	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB8033	CLADOSPORIUM HERBARUM (HORMODENDRUM) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB633	CLAMS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5657	CLOPIDOGREL (PLAVIX) RESPONSE GENOTYPING (CYP2C19)	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB257	CLOSTRIDIUM DIFFICILE EIA	ML MICROBIOLOGY	STERILE CONTAINER	Refrigerated. Must be received within 24 hours		PR
LAB253	CLOSTRIDIUM DIFFICILE TOXIN	ML MICROBIOLOGY	STERILE CONTAINER	Refrigerated. Must be received within 24 hours		PR
LAB5509A	CMV GUTHRIE CARD ANALYSIS	ML DNA	FILTER PAPER	RT		RT
LAB5156	CMV IGG	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB957	CMV IGM	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB5807	CMV PCR, QUALITATIVE, NOT BLOOD	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, Stool only must be in Star Buffer		PRTM
LAB913	CMV DNA QUANTITATIVE PCR	ML MOLECULAR MICROBIOLOGY	PEARL TOP or 2 EDTA	Centrifuge, aliquot, freeze		CAF
LAB5223	CO2, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB55	CO2, TOTAL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB378	COCAINE, URINE, QUALITATIVE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		AR
LAB1654	COCKLEBUR IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1675	COCKROACH, GERMAN IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1616	COCONUT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB602	CODFISH IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB6034	COMPREHENSIVE ENVIRONMENTAL PANEL	ML IMMUNOLOGY	SST-GOLD x2	Centrifuge, aliquot, freeze	7 mL of blood	AF
LAB17	COMPREHENSIVE METABOLIC PANEL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB5660	CONNEXIN 26 AND 30 DNA ASSAYS	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB26300	CONNEXIN 26 AND 30 GUTHRIE CARD	ML DNA	FILTER PAPER	RT		RT
LAB5658	CONNEXIN 26 DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB5659	CONNEXIN 30 DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB594	CORN IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB61	CORTISOL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
LAB1677	COTTON LINTERS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1636	COTTONWOOD (WHITE POPLAR) TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1618	CRAB IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB149	C-REACTIVE PROTEIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB383	CREATININE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB65	CREATININE, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB8029	CREATININE, URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB11090	CREATININE, URINE (ORGANIC ACIDS)	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, freeze		AF
LAB385	CREATININE, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate-include time and volume		CAR
LAB6011	CRYPTOCOCCAL ANTIGEN, CSF	ML IMMUNOLOGY	CSF TUBE	Unspun, Freeze		USPF
LAB779	CRYPTOCOCCAL ANTIGEN, SERUM	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB11098	CSF ALBUMIN (UNC)	ML CHEMISTRY	CSF TUBE	Centrifuge, aliquot, refrigerate		CAR
LAB11000	CSF IGG	ML CHEMISTRY	CSF TUBE	Centrifuge, aliquot, refrigerate		CAR
LAB195	CSF PROTEIN	ML CHEMISTRY	CSF TUBE	Centrifuge, aliquot,refrigerate		CAR
LAB6604A	CT NAA	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Hologic collection devices		PR
LAB627	CURVALARIA LUNATA IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB6079	CUSTOM SEQUENCING ANALYSIS	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5079	CYCLOSPORA/ISOSPORA EXAM	ML MICROBIOLOGY	STERILE CONTAINER	Parapak. Room Temp.		RT
LAB874	CYCLOSPORINE LEVEL	ML SPECIAL CHEMISTRY	LAVENDER	Whole blood, refrigerate		WBR
LAB5342	CYCLOSPORINE, 2 HOUR	ML SPECIAL CHEMISTRY	LAVENDER	Whole blood, refrigerate		WBR
LAB5343	CYCLOSPORINE, TROUGH	ML SPECIAL CHEMISTRY	LAVENDER	Whole blood, refrigerate		WBR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB542	CYSTIC FIBROSIS DNA PANEL	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5504A	CYTOGENETICS CANCER/FISH BLOOD ORDER	ML CYTOGENETICS	GREEN TOP-SODIUM HEPARIN	Whole Blood, Room Temperature		WBRT
LAB5504	CYTOGENETICS CANCER/FISH NON-BLOOD ORDER	ML CYTOGENETICS	OBTAIN FROM LAB - Bone Marrow collection	Room Temperature		RT
LAB5505	CYTOGENETICS POSTNATAL/FISH	ML CYTOGENETICS	OBTAIN FROM LAB - Tissue collection media	Room Temperature		RT
LAB5505A	CYTOGENETICS POSTNATAL/FISH, BLOOD	ML CYTOGENETICS	GREEN TOP-SODIUM HEPARIN	Whole Blood, Room Temperature		WBRT
LAB5501	CYTOGENETICS PRENATAL	ML CYTOGENETICS	OBTAIN FROM LAB - Multiple 15 ml Falcon screw	Room Temperature		RT
LAB5661	CYTOGENETICS TEST, OTHER	ML CYTOGENETICS	OBTAIN FROM LAB	Call the Laboratory for specific specimen requirements (984-974-1790)		RT
LAB595	D. FARINAE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB597	D. PTERONYSSINUS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5617	DABIGATRAN (ECARIN CHROMOGENIC ASSAY)	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB5617	DABIGATRAN (ECARIN CHROMOGENIC ASSAY)	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.		AF
LAB313	D-DIMER, QUANTITATIVE	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB5159	DIAGNOSTIC HIV AG/AB	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB5662	DIGEORGE FISH STUDY	ML CYTOGENETICS	GREEN TOP-SODIUM HEPARIN	Whole Blood, Room Temperature		WBRT
LAB23	DIGOXIN LEVEL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB8016	DNA EXTRACT AND HOLD	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB15721	DNA FINGERPRINTING, DONOR ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB57932	DNA FINGERPRINTING, POST-TRANSPLANT - CD3 ENRICHED	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB57932BM	DNA FINGERPRINTING, POST-TRANSPLANT BONE MARROW	ML DNA	LAVENDER	Bone marrow, unspun, refrigerate	2mL	P U R
LAB57932BL	DNA FINGERPRINTING, POST-TRANSPLANT BLOOD	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB16049	DNA FINGERPRINTING, PRE-TRANSPLANT ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB591	DOG DANDER IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB8923	EBV VIRAL LOAD (QUANTITATIVE)	ML DNA	LAVENDER	Centrifuge, Aliquot, Freeze		CAF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB588	EGG WHITE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	.5mL blood	AF
LAB589	EGG YOLK IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	.5mL blood	AF
LAB794	EHRlichia ANTIBODY PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB16	ELECTROLYTE PANEL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB608	ELM TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB8013	ENA PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB639	ENGLISH PLANTAIN IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5162	ENT ENVIRONMENTAL PANEL	ML IMMUNOLOGY	SST-GOLD x2	Centrifuge, aliquot, freeze	7mL of blood	AF
LAB5163	ENT FOOD PANEL	ML IMMUNOLOGY	SST-GOLD x2	Centrifuge, aliquot, freeze	3mL of blood	AF
LAB1333	ENTEROVIRUS PCR, BLOOD	ML MOLECULAR MICROBIOLOGY	SST-GOLD	Spin, Refrigerate		PR
LAB5245	ENTEROVIRUS PCR, CSF	ML MOLECULAR MICROBIOLOGY	CSF TUBE	Refrigerate		PR
LAB5166	EOHS HIV ANTIGEN/ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB629	EPICOCCUM PURPURASCENS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5242	EPSTEIN BARR VIRUS PCR, CSF	ML DNA	CSF TUBE	Refrigerated		PR
LAB5809	EPSTEIN BARR VIRUS PCR, OTHER	ML DNA	STERILE CONTAINER	Refrigerated		PR
LAB863	EPSTEIN-BARR VIRUS ANTIBODY PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB10673	EQUAL MIX, APTT	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB10672	EQUAL MIX, PT/INR	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB10674	EQUAL MIX, THROMBIN CLOTTING TIME	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB523	ESTRADIOL(ESTROGEN) LEVEL	ML CHEMISTRY	RED TOP -PLAIN	Centrifuge, aliquot, freeze		AF
LAB46	ETHANOL ENZYMATIC	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB5148	ETHANOL, GC	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB714	ETHYLENE GLYCOL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB875A	EVEROLIMUS	ML SPECIAL CHEMISTRY	LAVENDER	Whole blood, refrigerate		WBR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB5161	EXTRACTABLE NUCLEAR ANTIGEN	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	2 mL of blood	AF
LAB758	FACTOR 10 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB309	FACTOR 11 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB310	FACTOR 12 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB303	FACTOR 2 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB304	FACTOR 5 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB305	FACTOR 7 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB306	FACTOR 8 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB308	FACTOR 9 ACTIVITY	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB5668	FACTOR II (PROTHROMBIN) DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5319	FACTOR IX INHIBITOR	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB346	FACTOR V LEIDEN DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB11101	FACTOR VIII (VWF PANEL)	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB1116	FACTOR VIII INHIBITOR	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB5320	FACTOR XIII ASSAY	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB8028	FEATHER MIX IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB390	FECAL FAT, QUALITATIVE	ML URINALYSIS	STERILE CONTAINER	Unspun, refrigerate		USPR
LAB6052	FECAL LACTOFERRIN	ML MICROBIOLOGY	STERILE CONTAINER	Refrigerated. Specimen must be <72 hours old.		PR
LAB68	FERRITIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB287	FETAL FIBRONECTIN [1230100192]	ML CHEMISTRY or REX	FETAL FIBRONECTIN COLLECTION	refrigerated		PR
LAB5639	FETAL HEMOGLOBIN (FETOMATERNAL BLEED) BY FLOW CYTOMETRY	ML FLOW CYTOMETRY	LAVENDER	Whole blood, refrigerated, 96 hours from collection		WBR
LAB314	FIBRINOGEN	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.		AF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB1121	FIBRINOGEN ANTIGEN	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB5321	FIBRINOGEN IEP	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB761	Fibrin Degradation Products Screen (FDP)/ Fibrin Split Products	ML SPECIAL COAGULATION	FDP Tube – obtain from ML Coag or Spec Coag	Centrifuge, aliquot,freeze	.3 ml	AF
LAB12500	FII AND FV DNA PANEL	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5670	FLT3 DNA ANALYSIS	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB6739	FLT3 (ITD) & FLT3 (TKD) DNA PANEL	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5344	FLUCONAZOLE	ML SPECIAL CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB69	FOLATE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
LAB86	FOLLICLE STIMULATING HORMONE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
LAB738	FRAGILE X DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB6738	FRAGILE X METHYLATION STATUS	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB8024	FUSARIUM PROLIFERATUM (F MONOLIFORME) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB85	GAMMA GT	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB6604	GC NAA	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Hologic collection devices		PR
LAB28	GENTAMICIN LEVEL, PEAK	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB27	GENTAMICIN LEVEL, RANDOM	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB26	GENTAMICIN LEVEL, TROUGH	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB6053	GI PATHOGEN PANEL	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Aliquot into Cary Blair		ATM
LAB644	GIANT RAGWEED IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB571	GLUCOSE 6 PHOSPHATE DEHYDROGENASE	ML SPECIAL HEMATOLOGY	LAVENDER	Whole blood, refrigerate		WBR
LAB186	GLUCOSE, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB185	GLUCOSE, CSF	ML CHEMISTRY	CSF TUBE	Centrifuge, aliquot,refrigerate		CAR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB81	GLUCOSE, FASTING	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB82	GLUCOSE, RANDOM	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5192	GLUCOSE, RANDOM URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB395	GLUCOSE, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate, record time and volume		CAR
LAB610	GOLDENROD IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB8011	GOOSEFOOT (LAMB'S QUARTERS) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB9960	GOOSEFOOT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB622	GREEN PEA IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1773	GTT 1 HOUR	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB11500	GTT 1 HOUR PERFORMABLE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB1774	GTT 2 HOUR	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB11503	GTT 2 HOUR MOD PERFORMABLE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB11502	GTT 2 HOUR PERFORMABLE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB1767	GTT 3 HOUR	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB11504	GTT 3 HOUR PERFORMABLE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB1772	GTT FASTING	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB11511	GTT FASTING PERFORMABLE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB89	HAPTOGLOBIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB614	HAZELNUT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB752	HCG Quantitative Blood	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB101	HDL CHOLESTEROL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5167	HE4	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB5168	HELICOBACTER PYLORI ANTIBODIES	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB90	HEMOGLOBIN A1C	ML CHEMISTRY	LAVENDER	Whole Blood, Refrigerate		WBR
LAB5176	HEMOGLOBIN/THALASSEMIA PROFILE	ML SPECIAL HEMATOLOGY	LAVENDER	Whole blood, Refrigerate		WBR
LAB5179	HEMPATH LEUKEMIA FLOW	ML SPECIAL HEMATOLOGY	LAVENDER			
LAB5323	HEPARIN INDUCED THROMOCYTOPENIA PANEL	ML SPECIAL COAGULATION	RED TOP -PLAIN	Incubate at 37C for 2 hours. Centrifuge, Aliquot, and Freeze	min. 4 aliquots of min. 0.5 mL, using all serum	AF
LAB5324	HEPARIN PF4 ANTIBODY	ML SPECIAL COAGULATION	RED TOP -PLAIN	Centrifuge, Aliquot, and Freeze	min. 4 aliquots of min. 0.5 mL, using all serum	AF
LAB5325	HEPARIN-INDUCED PLATELET ANTIBODY	ML SPECIAL COAGULATION	RED TOP -PLAIN	Incubate at 37C for 2 hours. Centrifuge, Aliquot, and Freeze	min. 4 aliquots of min. 0.5 mL, using all serum	AF
LAB316	HEPARIN-LOW MOLECULAR WEIGHT	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.		AF
LAB317	HEPARIN-UNFRACTIONATED	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.		AF
LAB20	HEPATIC FUNCTION PANEL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB798	HEPATITIS A ANTIBODY, IGM	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
	HEPATITIS A ANTIBODY, TOTAL [1230101224]	REX	SST-GOLD	Freeze		AF
LAB6054	HEPATITIS A IGG	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB549	HEPATITIS B CORE ANTIBODY, IGM	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB1242	HEPATITIS B CORE ANTIBODY, TOTAL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB951	HEPATITIS B DNA, ULTRAQUANTITATIVE, PCR	ML MOLECULAR MICROBIOLOGY	SST-GOLD	Centrifuge, freeze		PF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB472	HEPATITIS B SURFACE ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		APR
LAB471	HEPATITIS B SURFACE ANTIGEN	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB868	HEPATITIS C ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB915	HEPATITIS C GENOTYPE	ML MOLECULAR MICROBIOLOGY	SST-GOLD	Centrifuge, freeze		PF
LAB887	HEPATITIS C RNA, QUANTITATIVE, PCR	ML MOLECULAR MICROBIOLOGY	SST-GOLD	Centrifuge, freeze		PF
LAB551	HEPATITIS PANEL, ACUTE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB833	HEREDITARY HEMOCHROMATOSIS DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB8193	HEXAG PHOSPHOLIPID APTT	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB6025	HHV-6 PCR, BLOOD	ML MOLECULAR MICROBIOLOGY	PEARL TOP -or 2 KEDTA	Centrifuge, aliquot, Refrigerate		AR
LAB6024	HHV-6 PCR, OTHER	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate		PR
LAB5326	HIGH MOLECULAR WEIGHT KININOGEN MIX	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB150	HIGH SENSITIVITY CRP	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5169	HIV ANTIGEN/ANTIBODY COMBO	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB878	HIV RNA, QUANTITATIVE, PCR	ML DNA	PINK TOP-KEDTA or 2 purple top	centrifuge,aliquot,freeze		CAF M,S IF EDTA
LAB869	HLA-B27 ANTIGEN	ML HLA	YELLOW (ACD), or GREEN TOP-SODIUM	Whole blood, Room Temp, 72 hours from collection		WBRT
LAB93	HOMOCYSTEINE	ML CHEMISTRY	LAVENDER	Centrifuge, aliquot, freeze		CAF
LAB635	HORSE DANDER IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB612	HOUSE DUST (GREER LABS) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB9216	HPV DNA HI RISK	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Thin Prep pap collection device		PRT
LAB5170	HSV ANTIBODIES, IGG	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB917	HSV PCR	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER OR SST GOLD	Refrigerate, swabs in UTM, biopsies in UTM, CSF-original container: Blood-		PR
LAB6055	HTLV I/II ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB73	IGA	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB74	IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB71	IGG	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB11099	IGG AND ALBUMIN FOR OLIGO EVAL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5663	IGH AND IKG CLONALITY ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB10970	IGH, IKG AND TCR GAMMA CLONALITY ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB72	IGM	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB174	IMMUNOFIXATION ELECTROPHORESIS	ML SPECIAL CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AR
LAB5193	IMMUNOFIXATION ELECTROPHORESIS, URINE	ML SPECIAL CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB735	IMMUNOGLOBULIN FREE LT CHAINS BLOOD	ML SPECIAL CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5327	INCUBATED APTT	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
	INSULIN, TOTAL [1230101287]	REX	RED TOP PLAIN	Freeze		AF
LAB94	IRON	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AR
	IRON + TIBC [1230100503]	REX	RED TOP PLAIN	refrigerated		AR
LAB9432	IRON PANEL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5150	IT MIX PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB6712A	ITRACONAZOLE LEVEL	ML SPECIAL CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB7500	JAK-2 (V617) MUTATION	ML DNA	EDTA x2 or YELLOW (ACD)	unspun		PR M,S IF EDTA
LAB7501	JAK-2 (V617F) QUANTITATIVE	ML DNA	EDTA x2 or YELLOW (ACD)	unspun		PR M,S IF EDTA
LAB1647	JOHNSON GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5664	KARYOTYPE CYTOGENETICS	ML CYTOGENETICS	GREEN TOP-SODIUM HEPARIN	Whole Blood, Room Temperature		WBRT
LAB603	KENTUCKY BLUE (MEADOW) GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB8002	L AND D HIV AG/AB COMBO	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB96	LACTATE DEHYDROGENASE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APRT
LAB188	LACTATE DEHYDROGENASE, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		ART
LAB1737	LATEX IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB102	LDL CHOLESTEROL, DIRECT	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB886	LEGIONELLA ANTIGEN, URINE	ML MICROBIOLOGY	STERILE CONTAINER	Refrigerated		PR
LAB1585	LENTIL IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
	LIDOCAINE LEVEL [1230101308]	REX	RED TOP PLAIN	refrigerated		AR
LAB99	LIPASE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5228	LIPASE, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB18	LIPID PANEL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB29	LITHIUM LEVEL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB5328	LMWH INDUCED PLATELET AGGREGATION	ML SPECIAL COAGULATION	RED TOP -PLAIN	Centrifuge, Aliquot, and Freeze	min. 4 aliquots of min. 0.5 mL, using all serum	AF
LAB642	LOBSTER IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5329	LUPUS INHIBITOR PANEL	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB87	LUTEINIZING HORMONE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
LAB5354	LYME DISEASE SEROLOGY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		APR
LAB5920	LYMPH MARKER COMPLETE, FLOW	ML FLOW CYTOMETRY	LAVENDAR	Whole blood, Room Temp, 48 hours		WBRT
LAB5922	LYMPH MARKER LIMITED, FLOW	ML FLOW CYTOMETRY	LAVENDAR	Whole blood, Room Temp, 48 hours		WBRT
LAB1646	MAEDOW FESCUE GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB103	MAGNESIUM	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5229	MAGNESIUM, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB5195	MAGNESIUM, RANDOM URINE	ML CHEMISTRY	STERILE CONTAINER	Unspun Aliquot,refrigerate		AR
LAB5194	MAGNESIUM, TIMED URINE	ML CHEMISTRY	STERILE CONTAINER	Unspun, Aliquot and refrigerate-note time and volume		USPR
LAB405	MAGNESIUM, URINE	ML CHEMISTRY	STERILE CONTAINER	Unspun, aliquot and refrigerate		USPR
LAB1641	MAPLE LEAF SYCAMORE TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB6039	MATERNAL CELL CONTAMINATION DNA ANALYSIS	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5665	MCAD DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB1054	METHADONE, URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB481	METHOTREXATE LEVEL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB5355	METHOTREXATE LEVEL, PEDIATRIC HIGH DOSE	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB7376	MGMT DNA ANALYSIS	ML DNA	SLIDE	Requires Paraffin block from AP		
LAB5667	MICROARRAY/KARYOTYPE 20 CELL	ML CYTOGENETICS	(1)GREEN TOP-SODIUM HEPARIN and (1)	Whole Blood, Room Temperature		WBRT
LAB5672	MICROARRAY/KARYOTYPE 5 CELL	ML CYTOGENETICS	(1)GREEN TOP-SODIUM HEPARIN and (1)	Whole Blood, Room Temperature		WBRT
LAB14512	MICROSATELLITE INSTABILITY DNA ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB586	MILK (COW) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB12600	MLH1 PROMOTER METHYLATION	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB482	MONONUCLEOSIS SCREEN	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		CAF
LAB1688	MOUSE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB8021	MPO ELISA PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB620	MUCOR RACEMOSUS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1658	MUGWORT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
	MUMPS IGG ANTIBODY [1230100066]	REX	SST-GOLD	refrigerated		APR
LAB5151	MUTTON IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB6733	MYELOID MUTATION PANEL - AML(BLOOD)	ML DNA	LAVENDER	Whole blood, refrigerate		WBR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB6735	MYELOID MUTATION PANEL - MDS (BLOOD)	ML DNA	LAVENDER	Whole blood, refrigerate		WBR
LAB6767	MYELOID MUTATION PANEL - MPN (BLOOD)	ML DNA	LAVENDER	Whole blood, refrigerate		WBR
LAB11097	MYELOMA SERUM CHEMISTRIES	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB11096	MYELOMA WORKUP, 24 HOUR URINE	ML SPECIAL CHEMISTRY	24 HR URINE CONTAINER	Centrifuge, Aliquot, refrigerate, record volume and time		CAR
LAB11088	MYELOMA WORKUP, SERUM	ML SPECIAL CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB11095	MYELOMA WORKUP, URINE	ML SPECIAL CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot, and refrigerate		CAR
LAB5720	MYELOPEROXIDASE ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB51A	NEONATAL BILIRUBIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5919	NEUTROPHIL FUNCTION	ML FLOW CYTOMETRY	LAVENDER	Whole blood, Room Temp, 72 hours from collection		WBRT
LAB6335	NOROVIRUS,PCR	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate; Send specimen within 2 days		PR
LAB5678	NPM1, QUANTITATIVE TYPE A	ML DNA	YELLOW (ACD) or EDTA x2	unspun		PR M,S IF EDTA
LAB643	OAT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5152	OB GYN CMV WITH REFLEX	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB11001	OLIGOCLONAL BAND EVALUATION	ML SPECIAL CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB416	OPIATE, URINE, QUALITATIVE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB417	OPIATE, URINE, QUANTITATIVE	ML SPECIAL CHEMISTRY	STERILE CONTAINER	Aliquot, refrigerate		AR
LAB1605	ORANGE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB623	ORCHARD GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB418	ORGANIC ACIDS, URINE	ML SPECIAL CHEMISTRY	STERILE CONTAINER	Aliquot and freeze		AF
LAB5230	OSMOLALITY, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB5109	OSMOLALITY, RANDOM URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB107	OSMOLALITY, SERUM	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		AR
LAB201	OSMOLALITY, STOOL	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB5108	OSMOLALITY, TIMED URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate, note volume and time		CAR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB420	OSMOLALITY, URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB1630	OYSTER IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB636	P CHRYSOGENUM (P. NOTATUM) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB6046	PARVOVIRUS PCR, BLOOD	ML MOLECULAR MICROBIOLOGY	LAVENDER	Whole blood, refrigerate		WBR
LAB6045	PARVOVIRUS PCR, OTHER	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, original container		PR
LAB592	PEANUT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1692	PECAN (HICKORY) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB1622	PECAN NUT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5153	PEDIATRIC FOOD PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze	3mL of blood	AF
LAB1528	PERENNIAL ALLERGEN PANEL IGE	ML IMMUNOLOGY	SST-GOLD x2	Centrifuge, aliquot, freeze		AF
LAB30	PHENOBARBITAL LEVEL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB32	PHENYTOIN LEVEL, FREE	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB31	PHENYTOIN LEVEL, TOTAL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, refrigerate		AR
LAB113	PHOSPHORUS	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB5231	PHOSPHORUS, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB427	PHOSPHORUS, URINE, RANDOM	ML CHEMISTRY	STERILE CONTAINER	Unspun, Aliquot and refrigerate		USAR
LAB426	PHOSPHORUS, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Aliquot and refrigerate-Note time and volume		AR
LAB1660	PIGWEEED, COMMON IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB248	PINWORM PREP	ML MICROBIOLOGY	TAPE PREP	RT		RT
LAB847	PLASMINOGEN ACTIVITY	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB5918	PNH ASSAY	ML FLOW CYTOMETRY	LAVENDER	Whole blood, Refrigerated, 72 hours from collection		WBR
LAB6732	PORCINE FACTOR VIII INHIBITOR ASSAY	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB1607	PORK IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5363	POSACONAZOLE	ML SPECIAL CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB11003	POST-DDAVP PANEL	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB114	POTASSIUM	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB193	POTASSIUM, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB577	POTASSIUM, STOOL	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB434	POTASSIUM, URINE, RANDOM	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB435	POTASSIUM, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate, record time and volume		CAR
LAB637	POTATO (WHITE) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB8020	PR3 ELISA PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5677	PRADER-WILLI AND ANGELMAN DNA ASSAYS	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB115	PREALBUMIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB11002	PRE-DDAVP PANEL	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB1114	PREKALLIKREIN SCREEN	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB5674	PRIMARY CILIARY DYSKINESIA (PCD) MUTATION TESTING	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5361	PRO-BNP	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB529	PROGESTERONE	ML CHEMISTRY	RED TOP -PLAIN	Centrifuge, aliquot, refrigerate		AF
LAB531	PROLACTIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		AF
LAB743	PROTEIN / CREATININE RATIO, URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,freeze		CAF
	PROTEIN / CREATININE RATIO, URINE [1230101604]	REX	STERILE CONTAINER	refrigerated		PR
LAB11094	PROTEIN 24 HOUR URINE	ML CHEMISTRY	24 HR URINE CONTAINER	Centrifuge, aliquot, refrigerate		CAR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB489	PROTEIN C ACTIVITY	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB119	PROTEIN ELECTROPHORESIS, SERUM	ML SPECIAL CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AR
LAB491	PROTEIN S ACTIVITY	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB492	PROTEIN S ANTIGEN, FREE	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB9685	PROTEIN S PANEL	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB196	PROTEIN, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB118	PROTEIN, TOTAL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB441	PROTEIN, URINE, 24 HOUR	ML CHEMISTRY	24 HR URINE CONTAINER	Centrifuge, aliquot,freeze-record time and volume		CAR
LAB439	PROTEIN, URINE, RANDOM	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,freeze		CAR
LAB5400	PROTEIN/CREATININE RATIO, 24 HOUR URINE	ML CHEMISTRY	24 HR URINE CONTAINER	Centrifuge, aliquot,freeze-record time and volume		CAR
LAB320	PROTIME-INR	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within 4 hours		AF
LAB116	PSA, DIAGNOSTIC	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
LAB5362	PSA, SCREENING	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
LAB10698	PTH	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, freeze		AF
LAB5902	QUANTIFERON TB GOLD	ML IMMUNOLOGY	Special Tube	Special tube. Do not spin. Deliver <16 hours		WBRT
LAB1661	RAGWEED, SHORT (COMMON) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5810	RAPID INFLUENZA PCR	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, UTM collection kit		PRTM
LAB5811	RAPID RSV/INFLUENZA PCR	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, UTM collection kit		PRTM
LAB885	RAPID STREP SCREEN	ML MICROBIOLOGY	STERILE SWAB	Swab. Room Temp		RT
LAB5154	REDTOP BENTGRASS	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB19	RENAL FUNCTION PANEL	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB1136	REPTILASE TIME	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.	min. 4 aliquots of min. 0.3 mL, using all plasma	AF
LAB1924	RESPIRATORY ALLERGY PANEL 2	ML IMMUNOLOGY	SST-GOLD x2	Centrifuge, aliquot, freeze	7 mL blood	AF
LAB5812	RESPIRATORY VIRUS GROUP NAA	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, UTM collection kit		PRTM
LAB296	RETICULOCYTES	ML HEMATOLOGY	LAVENDER	Whole blood, Refrigerated: 24 Hours		WBR
LAB206	RHEUMATOID FACTOR	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1601	RHIZOPUS NIGRICANS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB624	RICE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1265	RICKETTSIA (RMSF) ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate	1 mL of blood	PR
LAB5921	RITUXIN CD20, FLOW	ML FLOW CYTOMETRY	LAVENDAR	Whole blood, Room Temp, 48 hours		WBRT
LAB12501	RNA EXTRACT AND HOLD	ML DNA	YELLOW (ACD) or EDTA x2	unspun,		PR M,S IF EDTA
LAB494	RPR	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB496	RUBELLA ANTIBODY, IGG	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
	RUBEOLA ANTIBODY IGG [1230101422]	REX	SST-GOLD	refrigerated		APR
LAB1649	RYE GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB34	SALICYLATE LEVEL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, Aliquot, Refrigerate		AR
LAB1624	SALMON IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1625	SESAME SEED IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB8025	SETOMELANOMMA ROSTRATA (H. HALODES) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1666	SHEEP SORREL IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB8031	SHELLFISH PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB617	SHRIMP IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB339	SICKLE CELL SCREEN	ML SPECIAL HEMATOLOGY	LAVENDER	Whole Blood, Refrigerate		WBR
LAB875	SIROLIMUS LEVEL	ML SPECIAL CHEMISTRY	LAVENDER	Whole blood, refrigerate		WBR
LAB122	SODIUM	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB197	SODIUM, BODY FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB198	SODIUM, STOOL	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB444	SODIUM, URINE, RANDOM	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate		CAR
LAB445	SODIUM, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot,refrigerate, record time and volume		CAR
LAB8977	SOLID TUMOR MUTATION PANEL	ML DNA	SLIDE	Requires Paraffin block from AP		
LAB587	SOYBEAN IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB6035	STANDARD FOOD PANEL	ML IMMUNOLOGY	SST-GOLD x2	Centrifuge, aliquot,freeze		AF
LAB609	STRAWBERRY IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1578	SWEET POTATO IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1651	SWEET VERNAL IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB136	T3	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
LAB137	T3, FREE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB126	T4	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB127	T4, FREE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB5366	TACROLIMUS LEVEL, TIMED	ML SPECIAL CHEMISTRY	LAVENDER	Whole blood, refrigerate		WBR
LAB5367	TACROLIMUS LEVEL, TROUGH	ML SPECIAL CHEMISTRY	LAVENDER	Whole blood, refrigerate		WBR
LAB5679	T-CELL RECEPTOR CLONALITY ASSAY	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB124	TESTOSTERONE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		AF
	THEOPHYLLINE [1230100258]	REX	RED TOP PLAIN	refrigerated		AR
LAB324	THROMBIN TIME	ML COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor. Aliquot and Freeze within four hours.		AF
LAB5771	THYROID FUNCTION CASCADE	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot,refrigerate		APR
LAB858	THYROID PEROXIDASE ANTIBODY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze <8 hrs after collection	.5mL blood	AF
LAB1652	TIMOTHY GRASS IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB723	TISSUE TRANSGLUTAMINASE, IGA	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB36	TOBRAMYCIN LEVEL, PEAK	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB38	TOBRAMYCIN LEVEL, TROUGH	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot,refrigerate		AR
LAB625	TOMATO IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB11102	TOTAL PROTEIN FOR SPE	ML CHEMISTRY	SST-GOLD	Centrifuge, Aliquot, Refrigerate		APR
LAB500	TOXICOLOGY SCREEN, URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot, Refrigerate		CAR
LAB501	TOXOPLASMA GONDII ANTIBODY, IGG	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB659	TOXOPLASMA GONDII ANTIBODY, IGM	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB5680	TOXOPLASMA PCR, BLOOD	ML MOLECULAR MICROBIOLOGY	PEARL TOP - or 2 KEDTA	Centrifuge, aliquot,Refrigerate		AR
LAB5813	TOXOPLASMA PCR, NOT BLOOD	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate		PR
LAB5172	TP-PA	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		PR
LAB133	TRANSFERRIN	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, refrigerate		APR
LAB8027	TREE NUT PANEL	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
	TREPONEMA PALLIDUM (SYPHILIS) [1230101476]	REX	RED TOP PLAIN	refrigerated		AR
LAB6696	TRICHOMONAS NAAT	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Hologic collection devices		PR
LAB1380	TRICHOMONAS SCREEN	ML MICROBIOLOGY	STERILE CONTAINER	Swab. Room Temp		RT
LAB5173	TRICHOPHYTON RUBRUM IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB134	TRIGLYCERIDES	ML CHEMISTRY	SST-GOLD	Centrifuge, Aliquot, Refrigerate		APR
LAB200	TRIGLYCERIDES, BODY FLUIDS	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot, Refrigerate		CAR
LAB747	TROPONIN I	ML CHEMISTRY	SST-GOLD	Centrifuge, Aliquot, Refrigerate		APR
LAB129	TSH	ML CHEMISTRY	SST-GOLD	Centrifuge, Aliquot, Refrigerate		APR
LAB640	TUNA IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1627	TURKEY (MEAT) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5681	UGT1A1 DNA ANALYSIS	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB5234	UREA NITROGEN, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot , Refrigerate		CAR

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB5189	UREA NITROGEN, TIMED URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot , Refrigerate, Record Time and Volume		CAR
LAB748	UREA NITROGEN, URINE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot , Refrigerate		CAR
LAB141	URIC ACID	ML CHEMISTRY	SST-GOLD	Centrifuge, Aliquot, Refrigerate		APR
LAB5235	URIC ACID, FLUID	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot , Refrigerate		CAR
LAB5401	URIC ACID, RANDOM URINE	ML CHEMISTRY	STERILE CONTAINER	Unspun, Aliquot and Refrigerate		USAR
LAB450	URIC ACID, URINE, TIMED	ML CHEMISTRY	STERILE CONTAINER	Unspun, Aliquot and Refrigerate, Record Time and Volume		USAR
LAB11091	URINE PROTEIN ELECTROPHORESIS (UPE)	ML SPECIAL CHEMISTRY	STERILE CONTAINER	Centrifuge, Aliquot, and refrigerate		CAR
LAB11092	URINE TOTAL PROTEIN FOR UPE	ML CHEMISTRY	STERILE CONTAINER	Centrifuge, aliquot, refrigerate		CAR
LAB24	VALPROIC ACID LEVEL, TOTAL	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, Aliquot, Refrigerate		AF
LAB41	VANCOMYCIN, PEAK	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, Aliquot, Refrigerate		AR
LAB40	VANCOMYCIN, RANDOM	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, Aliquot, Refrigerate		AR
LAB39	VANCOMYCIN, TROUGH	ML CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, Aliquot, Refrigerate		AR
LAB6047	VARICELLA (VZV) PCR, OTHER	ML MOLECULAR MICROBIOLOGY	STERILE CONTAINER	Refrigerate, swabs in UTM, biopsies in UTM, CSF-original container		PRTM
LAB162	VARICELLA ZOSTER ANTIBODY, IGG	ML IMMUNOLOGY	SST-GOLD	Centrifuge, Refrigerate		AF
LAB5685	VARICELLA ZOSTER PCR, BLOOD	ML MOLECULAR MICROBIOLOGY	LAVENDER	Whole blood, refrigerate		WBR
LAB859	VDRL, CSF	ML IMMUNOLOGY	CSF TUBE	Refrigerate		PR
LAB765	VISCOSITY, SERUM	ML SPECIAL HEMATOLOGY	RED TOP -PLAIN 3 FULL RED TOP TUBES	Centrifuge, aliquot, Refrigerate		AR
LAB67	VITAMIN B12	ML CHEMISTRY	SST-GOLD	Centrifuge, aliquot, Refrigerate		AF
LAB535	VITAMIN D 25 HYDROXY	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot, Refrigerate		APR
LAB1112	VON WILLEBRAND PANEL	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor Plasma. Aliquot and freeze within 4 hours	Min. 4 aliquots of min. 0.3mL. Using all plasma	AF
LAB757	VON WILLEBRAND ANTIGEN	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor Plasma. Aliquot and freeze within 4 hours	Min. 4 aliquots of min. 0.3mL. Using all plasma	AF
LAB5337	VON WILLEBRAND ACTIVITY	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor Plasma. Aliquot and freeze within 4 hours	Min. 4 aliquots of min. 0.3mL. Using all plasma	AF
LAB1111	VON WILLEBRAND MULTIMERS	ML SPECIAL COAGULATION	LT BLUE TOP-CITRATE	Platelet Poor Plasma. Aliquot and freeze within 4 hours	Min. 4 aliquots of min. 0.3mL. Using all plasma	AF

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB5708	VORICONAZOLE	ML SPECIAL CHEMISTRY	ROYAL BLUE TOP PLAIN	Centrifuge, aliquot, Refrigerate		AR
LAB1642	WALNUT (TREE POLLEN) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB604	WALNUTS (FOOD) IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB590	WHEAT IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1632	WHITE ASH TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB1639	WHITE OAK TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB621	WHITE PINE TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5155	WILLOW TREE IGE	ML IMMUNOLOGY	SST-GOLD	Centrifuge, aliquot,freeze		AF
LAB5926	HLA-ANTIBODY SCREEN	ML HLA	RED TOP - PLAIN	Room Temp, 72 hours from collection		USRT
LAB5928	HLA-MATCHED PLATELET ANTIBODY SCREEN	ML HLA	YELLOW (ACD) and RED TOP PLAIN	Room Temp, 24 hours from collection		USRT
LAB6026	HLA DSAPT	ML HLA	RED TOP - PLAIN	Room Temp, 72 hours from collection		USRT
LAB8151	HLA FSAB CL1 SPECIFICITY	ML HLA	RED TOP - PLAIN	Room Temp, 72 hours from collection		USRT
LAB8152	HLA FSAB CL1 SPECIFICITY	ML HLA	RED TOP - PLAIN	Room Temp, 24 hours from collection		USRT
LAB6028	HLA FLOW XM DONOR	ML HLA	2 YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB6028	HLA FLOW CROSSMATCH RECIPIENT	ML HLA	2 YELLOW (ACD) and RED TOP PLAIN	Room Temp, 24 hours from collection		WBUSRT
LAB5925	HLA DISEASE ASSOC WKUP (Must specify disease on order)	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8208	HLA CL I & II LOW RES TYPING	ML HLA	2 YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8059	HLA LOW RES AB TYPING	ML HLA	2 YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8115	HLA LOW RES C TYPING	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8139	HLA LOW RES DR TYPING	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8137	HLA LOW RES DQ TYPING	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8056	HLA LOW RESOLUTION DNA ABC	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8057	HLA LOW RESOLUTION DNA ABDR	ML HLA	2 YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8138	HLA LOW RESOLUTION DNA DR/DQ	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8062	HLA LOW RESOLUTION CLASS IA	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8081	HLA LOW RESOLUTION CLASS IB	ML HLA	YELLOW (ACD)	Room Temp, 24 hours from collection		WBUSRT
LAB8257	HLA HI RES SBT ABCDR	ML HLA	2 YELLOW (ACD) or 3 BUCCAL SWABS	Room Temp, 24 hours from collection		WBUSRT
LAB8258	HLA HI RES SBT A	ML HLA	YELLOW (ACD) or 3 BUCCAL SWABS	Room Temp, 24 hours from collection		WBUSRT
LAB8259	HLA HI RES SBT B	ML HLA	YELLOW (ACD) or 3 BUCCAL SWABS	Room Temp, 24 hours from collection		WBUSRT
LAB8260	HLA HI RES SBT C	ML HLA	YELLOW (ACD) or 3 BUCCAL SWABS	Room Temp, 24 hours from collection		WBUSRT
LAB8262	HLA HI RES SBT DRB1	ML HLA	YELLOW (ACD) or 3 BUCCAL SWABS	Room Temp, 24 hours from collection		WBUSRT

EPIC TEST ID	TEST DESCRIPTION	LAB SECTION	TUBE	Processing Requirements	Minimum Volume, where applicable	Label Directions -A Aliquot, P primary, AP aliquot or primary, TM transport media, WB, US unspun. R Refrigerate. RT Room Temp, F Freeze, C=Centrifuge M,S if EDTA = requires MULTIPLE, SEPARATE tubes
LAB8261	HLA HI RES SBT DQ	ML HLA	YELLOW (ACD) or 3 BUCCAL SWABS	Room Temp, 24 hours from collection		WBUSRT
LAB8258A	HLA HI RES SBT DP	ML HLA	YELLOW (ACD) or 3 BUCCAL SWABS	Room Temp, 24 hours from collection		WBUSRT
LAB67891	GLIOMA MUTATION PANEL	ML DNA	SLIDE	Requires Paraffin block from AP		
LAB67801	IDH1 AND IDH2 MUTATION TESTING	ML DNA	SLIDE	Requires Paraffin block from AP		
LAB7505	KIDNEY GENETICS MUTATION PANEL	ML DNA	YELLOW (ACD) or EDTA x2	Unspun, Refrigerated		WBR M,S IF EDTA
LAB6734	LYMPHOID MUTATION PANEL	ML DNA	LAVENDER	Whole blood, refrigerate		WBR
LAB6736	TERT PROMOTER MUTATION ANALYSIS	ML DNA	SLIDE	Requires Paraffin block from AP		